

8890-300 AUTOMATIC EXHAUST VALVE ADAPTER KIT

DESCRIPTION

The Amron International 8890-300 Automatic Exhaust Valve Adapter Kit provides automatic exhaust of excess O₂ from the treatment hood regardless of the input flow of oxygen.

It will track the supply flow and increase or decrease the flow as necessary. In the event of a loss of oxygen supply, the exhaust regulator will shut down.

OVERVIEW

Features:

- Connects to Oxygen Treatment Panel Models:
 - 8890 (Single Patient)
 - 8890-2 (Dual Patient)
- Automatic exhaust of excess O₂
- Tracks Supply Flow and increases or decreases flow as necessary
- Automatic shut down in the event of loss of oxygen supply

INSTRUCTIONS

1. Remove the 22mm white Teflon coupler (P/N 8890-004) & 9/16" internal star washer from the "EXHAUST" side of Model 8890 (shown), or "EXHAUST" sides 1 and 2 for Model 8890-2.
2. Clean exposed threads on the "EXHAUST" side of Model 8890 (shown), or "EXHAUST" sides 1 and 2 for Model 8890-2.
3. Install 9/16" internal star washer over exposed threads and tighten the white Teflon coupler (P/N 8890-014) located on Exhaust Regulator Adapter Kit using a 3/4" open-end wrench.
4. Push on 22mm flex tubing from hood or mask exhaust to elbow located on Exhaust Regulator Adapter Kit.

