

BASE STATION & REMOTE HEADSET PROGRAMMING



BASE STATION & REMOTE HEADSET PROGRAMMING

CLEARING THE BASE-STATION

When programming a replacement headset, the base-station needs to be cleared before a “Remote” unit can be linked to it. Follow the steps below to clear the base-station. All remote units should be “OFF” before starting this process.

1. Remove base-station from the Amron Communicator lid. Channel selection buttons “1” and “2” are located on the rear side of base-station.
2. Turn Amron Communicator Power “ON”. Located on the bottom right of the HUB, turn the base-station dial switch “ON”. The Green LED, located on the left side of base-station next to power jack, will turn on.
3. Press the base-station #1 programming button until the Red LED starts flashing.
4. Press the base-station #1 programming button 7 times. The Red LED will begin rapid flashing, and then will go out. At this point, the base-station has no remotes programmed to channel 1.
5. Press the base-station #2 programming button until the Red LED starts flashing.
6. Press the base-station #2 programming button 7 times. The Red LED will begin rapid flashing, and then will go out. At this point, the base-station has no remotes programmed to channel 2.
7. The headsets channels are now cleared of programing to the base-station.

Follow the “Base-Station Programming Instructions” to link headsets.

BASE STATION & REMOTE HEADSET PROGRAMMING

BASE-STATION PROGRAMMING INSTRUCTIONS

1. All "Remote" headsets to be fully charged and in the "OFF" position.
2. Remove base-station from the Amron Communicator lid. Channel selection buttons "#1" and "#2" are located on the rear side of base-station.
3. Turn Amron Communicator power "ON". Located on the bottom right of the HUB, turn the base-station dial switch "ON" and adjust it to a middle volume range. The Green LED, located on the left side of base-station next to power jack, will turn on.
4. Press and hold the base-station's programming button "#1" (to program to channel #1) or "#2" (to program to channel #2) until the Red LED begins to flash rapidly. This indicates that the base-station is ready to link to a Remote Headset.
5. Press the Power button on the Remote Headset and release. The Power button should be flashing blue, and ready to be programmed.
6. Press REMOTE Headset volume DOWN arrow 3 times and release. Immediately carry out Step 7.
7. Press and HOLD the volume UP arrow until the Blue LED begins flashing, then release. Within a few seconds, the Blue LED will turn off and then back on when the pairing process is complete.
8. In order to program additional headsets, power down all "Remote" headsets by holding the power button down for at least 2 seconds. The LED on the "base-station" will go out. Press and hold the programming button of the channel you wish to program until the Red LED begins flashing, then repeat steps 5 and 7 for each additional headset.

Each base-station can have up to 3 remote headsets linked to each channel.