HANS RUDOLPH, INC. 7450 SERIES V2 MASK™
REUSABLE with MASK ADAPTER
INSTRUCTIONS FOR USE

Figure 1

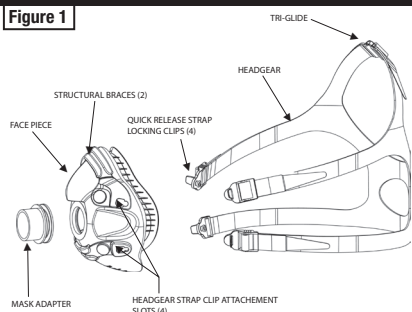
Figure 2


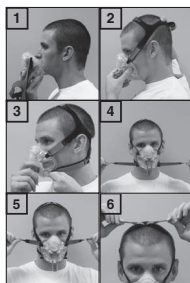
Figure 3

Figure 4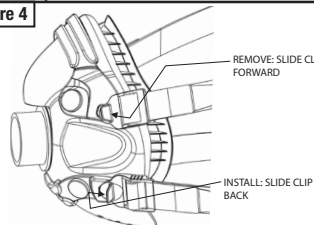

Intended Use:

The 7450 series face mask is a reusable, multi-patient, multi-use mask intended for use with mask adapters for connection with flow measuring instruments used in lung function testing procedures with spontaneous breathing adults (>30 kg) in hospitals and other clinical settings by individuals that have at least minimal training or instructions on the use of the mask as well as the devices to which the mask is intended to connect. Applications are for pulmonary function testing, exercise testing and respiratory research. The masks, adapters and headgear are supplied clean, non-sterile.

Caution:

Federal law restricts this device to sale by or on physician's order.

Warnings:

1. Shipped clean, non-sterile. If disinfected or sterile use is required, follow the procedures in this document prior to use of the mask.
2. To avoid skin irritation, never use any cleaning solutions or chemicals on this mask or headgear other than a mild detergent and water. Discontinue use if irritation or allergic reaction develops due to the mask.
3. Patients with facial hair may experience leakage
4. Do not use these masks and or adapters in a life support system such as a ventilator circuit.
5. Do not clean using an automated washer-disinfector as this process may remove device markings or finishes.
6. Frequency of cleaning, methods of cleaning or the use of cleaning agents, other than those specified in this document, or exceeding the number of processing cycles can have an adverse effect on the mask and consequently the safety or the quality of the therapy.
7. Mask fall into the semi-critical device category based on potential risk of infection. Although sterilization is recommended whenever practical, high level disinfection should be performed between uses on different patients.

Materials

1. Face Piece is silicone rubber with polycarbonate plastic structural braces for headgear strap clip attachments
2. Mask Adapter is silicone rubber
3. Headgear is a laminate of polyurethane foam with nylon fabrics
4. Headgear Tri-Glide and Strap Clips are polypropylene plastic

Recommend Headgear Sizes (L, M & S)

Mask Size (Headgear Size): L(L) M(M) S(S) XS(S) P(S)

Deadspace ml (Total Mask & Mask Adapter)

Mask Size (Deadspace ml): L(160) M(142) S(116) XS(105) P(95)

Service Life:

Mask, mask adapter and headgear when maintained and reprocessed per these instructions are expected to stay in service for 25 disinfection or sterilization cycles and the headgear 25 cleaning cycles. Visual inspection of the components during reprocessing will identify any defects. If defects are identified dispose of and do not use.

Operational Environment, Mask Assembly & Headgear:

1. Storage/Operational: Temperature: 5 – 40 °C, Humidity: 0 – 95% RH
2. Extreme Ambient Transport Conditions (Temporary): -40 – 70 °C, Humidity: 0 – 95% RH

Safety Information:

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SYMBOLS	
Medical device manufacturer	Needs protection from UV light sources
Do not use if the package is damaged or opened	Temperature limitations
Use-by date (first use)	Consult the instructions for use
Manufacturer's batch/lot code	European conformity (CE) mark
Manufacturer's catalogue/part number	Indicates product is a medical device
Date of the manufacture	Model (Series) number
For sale by/on physician's order only (USA)	Size large
Size medium	Size small
Size extra small	Size petite
United Kingdom conformity mark	

Mask Components (fig 1):

1. Mask face piece has plastic braces with slot openings for attachment of the headgear strap quick-release clips
2. Headgear has adjustable tri-glide on top of head and quick-release connection clips with strap locking adjustments
3. Mask Adapter is a flexible silicone rubber or rigid plastic designed to attach to flow meters

Application & Fitting (fig 2):

1. Determine patient mask size with optional Mask Sizing Gauge (fig 3)
2. Place mask over patient's nose and mouth (1). Chin should fit into mask chin cup portion with top of mask sealing area on the bridge of the nose.
3. Slide headgear over patient's head (2) with one of the lower quick release strap clips disconnected. Reconnect headgear strap clip after headgear is mounted (3). Bottom straps should be positioned below ears and top straps above ears and below eyes.
4. Adjust headgear strap tension by pulling slightly away from the face on strap end hook sections until tension is evenly distributed at all four straps with mask snug but comfortable (4 & 5). The strap clips lock and hold the position of the strap. To loosen the straps pull up on the back of the clip. Attach hook ends to the headgear fabric to keep the ends attached and out of the way. Adjust straps at crown of head by pulling each in opposite direction until reaching the required tension then connect the Velcro™ (6).
5. Push the flow meter patient side into the port opening in the mask adapter.

Disassembly for Cleaning:

1. Separate headgear from mask.
2. Separate mask adapter from mask
3. Do not remove the upper and lower plastic braces (headgear attachment) from the mask facepiece

Cleaning Mask face piece, Mask Adapter & Headgear:

Manual Cleaning (Mask & Mask Adapter):

1. Soak components for 5 minutes in warm water with mild detergent (neutral pH).
2. Hand wash components with sponge or soft brush; then rinse with warm tap water.

- Place components in warm water bath and agitate for two minutes.
- Rinse in clean tap water for at least 1 minute.
- Allow components to air dry or dry with clean, lint free cloth.

Manual Cleaning (Headgear):

- Submerge headgear in warm soapy tap water; gently rub all areas.
- Rinse in clean tap water for 1 minute or until all soap is removed; air dry.
- Precaution: Do not use bleach, chlorine or alcohol based solutions to clean mask or headgear. These solutions can damage components. Direct sunlight exposure on mask and headgear components can cause deterioration and reduce product life.

High Level Disinfection (Mask & Mask Adapter less Headgear)

Cidex Liquid Glutaraldehyde	Steris Revital-Ox Resert	Hot Water Pasteurization
Thoroughly clean mask prior to disinfection or sterilization		
Soak mask components as per Cidex instructions	Soak mask components as per Steris instructions	Immerse mask components in hot water temperatures of 71.1°C to 76.6°C for 30 minutes
Rinse 1: Soak mask components in 1500 ml of sterile water for ≥ 1 minute		
Rinse 2: Soak mask components in 1500 ml of sterile water for ≥ 1 minute		
Dry mask components with clean lint free cloth (preferably sterile)		
Visually inspect all components for cleanliness, function and defects. If there are any residues, stains or organic debris then repeat the previous steps. Dispose of and replace all defective parts.		

Sterilization (Face Mask assembly less Headgear and Mask Adapter)

- Thoroughly clean mask prior to disinfection or sterilization
- Mask should be completely assembled (less headgear and mask adapter).

Steam Sterilization

Parameter	Steam (Pre-Vacuum)	Steam (Gravity Displacement)
Temperature	132.2 +3/-1°C	132.2 +3/-1°C
Sterilization Time	4 minutes	30 minutes
Dry Time	10 minutes	10 minutes
Packaging	Double pouched/wrapped in CSR	Tyvek® sterilization pouch

Steris V-Pro Low Temperature Sterilization

Sterilizer	Cycle
V-PRO® 1 Low Temperature Sterilization System	Standard Cycle
V-PRO® 1 Plus Low Temperature Sterilization System	Lumen and Non Lumen Cycles
V-PRO® maX Low Temperature Sterilization System	Lumen, Non Lumen and Flexible Cycles
V-PRO® 60 Low Temperature Sterilization System	Lumen, Non Lumen, and Flexible Cycles

Devices must be cleaned, rinsed and dried in accordance with the instruction for use of the sterilizers. Refer to STERIS's Operator Manual for proper operation of the unit and cycle selection.

Inspection:

Visually inspect all components for cleanliness, function and defects. If there are any residues, stains or organic debris then repeat the previous steps. Dispose of all defective parts.

Reassemble:

- Install mask adapter into face mask port opening with the mask adapter flanges placed on the inside and outside of the mask port opening. This may require some stretching and pulling to completely install the mask adapter such that the flanges are fully engaged and not disfigured.
- Mount headgear to the face piece with the snap-in quick release clips.

Functional Check:

- Mask adapter flanges should be stretched to fit snug without disfiguration on either side of the mask port opening. Do not use if the mask adapter is not completely engaged in the mask port opening.
- For function test hold palm of hand over the outlet of the mask adapter and create a slight positive pressure inside the mask by exhaling gently. If you detect a leak, reposition the mask or adjust straps tension. If leak continues regardless of adjustments try the next smaller size mask.
- Push the patient port side of the flow meter into the mask adapter. It should fit snug and not fall out without significant pulling.
- Do not use if it is non-functional

Storage:

- Components should be completely dry prior to storage in a sealed poly bag
- Label poly bag with disinfection/clean status, mask description, date and initials.

Recommendations for Disposal:

Treat as conventional solid waste in accordance with local and federal regulations.

Credits:

Velcro™ is a Trademark of Velcro, U.S.A.
 Cidex™ is a Trademark of Johnson & Johnson Medical Products, Inc.
 Tyvek® is a Trademark of DuPont
 Steris® is a Trademark of Steris

HANS RUDOLPH, Inc.
 8325 Cole Parkway, Shawnee, KS 66227 USA
 Tel: (913) 422-7788, Toll Free: (800) 456-6695, Fax: (913) 422-3337
 Website: www.rudolphkc.com, Email: hri@rudolphkc.com

E C Rep Ltd.
 5 Fitzwilliam Square East, Dublin 2, D02 R744, Ireland
 Tel: +353 1 2 544 944, Email: info@ecrep.ie

Swiss AR Services AG.
 Industriestrasse 47
 CH-6300 Zug

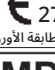
Responsible Person in the United Kingdom
 Responsible Person Ltd
 Summit House, 4-5 Mitchell Street, Edinburgh, EH6 78D, Scotland

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Velcro™ on Velcro, U.S.A.:n tavaramerkki
Cidex™ on Johnson & Johnson Medical Products, Inc.:n tavaramerkki