

HANS RUDOLPH inc.



MASK SIZING GAUGE

CHECK ADOBE PRINT SETTINGS BEFORE PRINTING

PAGE SCALING: NONE

IF PREVIEW IS OUTSIDE PRINTING BORDER

SELECT AUTO-ROTATE & CENTER

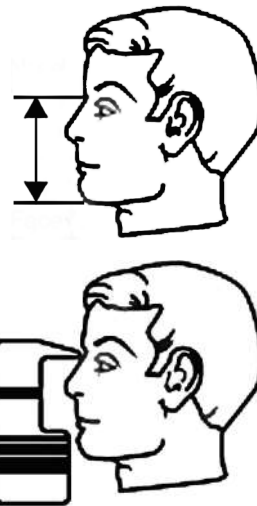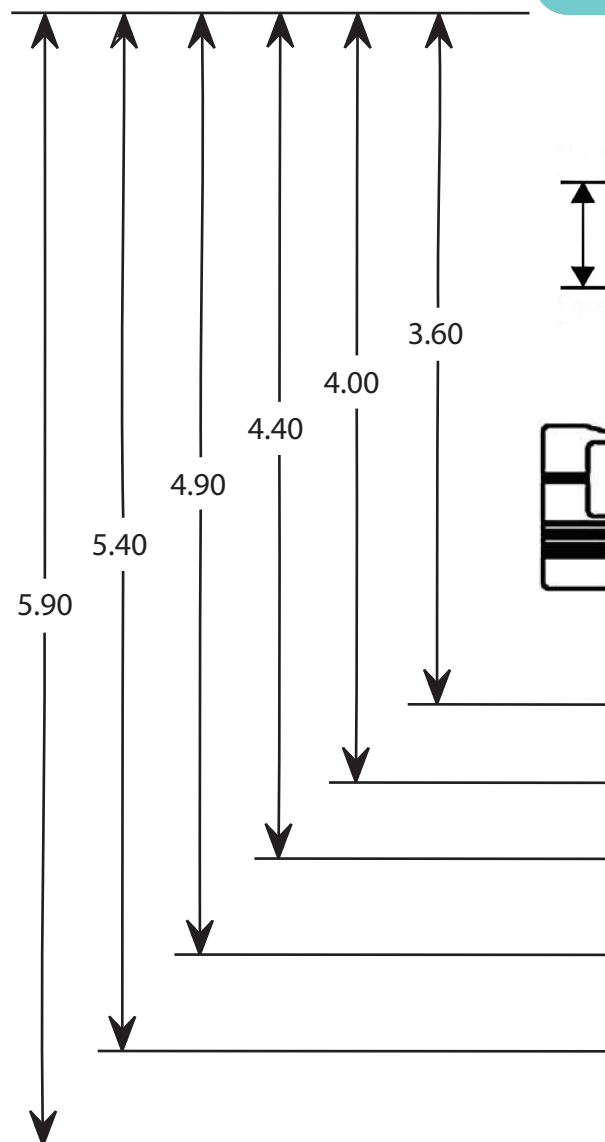

MASK SIZING GAUGE

Directions: Length measurements are used to determine the size of the Hans Rudolph Oro-Nasal mask that will best fit the patient. For measurement: the patient's facial muscles should be relaxed and jaw closed. Place the Gauge tip to the deepest point of the nasal root depression (sellion) and note the size indication where the bottom of the chin exits the gauge. The gray band widths indicate the size ranges.

This gauge is a reference to aid in sizing and not an exact measurement as other factors may need to be taken into account.

IF BETWEEN SIZES, WE RECOMMEND YOU USE THE SMALLER SIZE.

HANS RUDOLPH, inc.

PETITE (P)

EXTRA-SMALL (XS)

SMALL (S)

MEDIUM (M)

LARGE (L)

www.rudolphkc.com